



**MILLENNIUM
SURGICAL**

SURGICALINSTRUMENTS.COM


UNMATCHED SERVICE · UNLIMITED INSTRUMENTS

822 Montgomery Ave #205
Narberth, PA 19072
(800) 600-0428

Flash-Guard®

Sterilizer to Point of Use Transport System

- Dependable!** No moving parts during sterilization.
- Economical!** No expensive valves or filters to check, replace or maintain.
- Durable!** All stainless steel construction.
- Reliable!** Exhaust design works in unison with sterilizer.
- Universal!** Works in Pre-Vacuum or Gravity sterilizers.

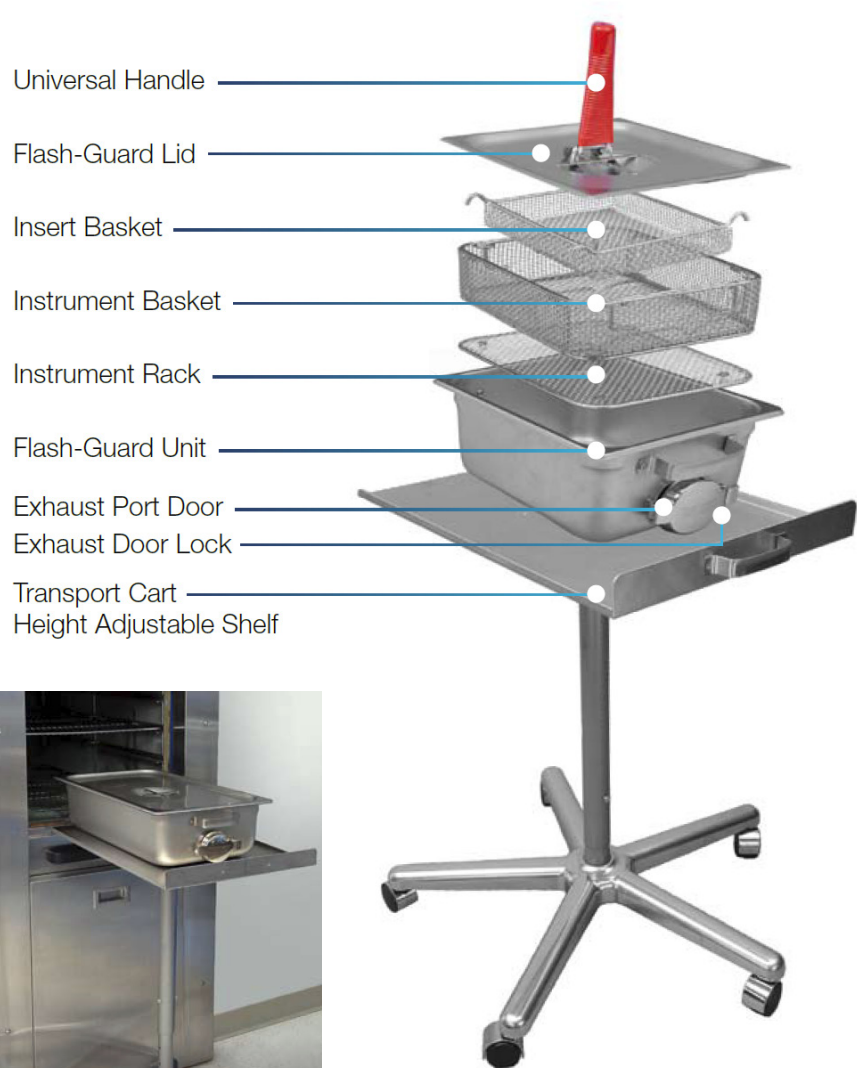

No lifting, burns or back injury.



Transports easily from sterilizer to point of use.



**MILLENNIUM
SURGICAL**

SURGICALINSTRUMENTS.COM

UNMATCHED SERVICE · UNLIMITED INSTRUMENTS

822 Montgomery Ave #205
Narberth, PA 19072
(800) 600-0428

Flash-Guard®

Sterilizer to Point of Use Transport System



Load

Place clean, decontaminated items into the basket. Place the basket into the Flash-Guard® container. The Instrument Rack should be used when sterilizing large items or specialized instrument trays.



Monitor

Place approved Level 5 or 6 Integrator inside the exhaust port for easy removal and inspection after sterilization cycle.



Sterilize

To sterilize unwrapped items using the Flash-Guard system in a Pre-Vacuum or Gravity sterilizer, OPEN EXHAUST PORT fully. Slide lid forward 3 inches for sterilization parameters to be obtained.



Transport

At completion of sterilization cycle, use the universal handle to slide the Flash-Guard lid closed. Remove Integrator from exhaust port and position on top of Flash-Guard lid and Biological Indicator if needed. Aggressively close Exhaust Port Door and pull Flash-Guard Unit onto Transport Cart using universal handle. Transport Flash-Guard system to point of use.

Flash-Guard® system designed to sterilize items using the "unwrapped method," for "immediate use," and be transported from the sterilizer to point of use. As with any sterilization process, the manufacturer advises strict adherences to AAMI, AORN Standards of Practice, OSHA Guidelines, and the JCAHO, CMA, AAHC regulatory agencies.



**MILLENNIUM
SURGICAL**

SURGICALINSTRUMENTS.COM

UNMATCHED SERVICE · UNLIMITED INSTRUMENTS

822 Montgomery Ave #205
Narberth, PA 19072
(800) 600-0428

Flash-Guard®

Sterilizer to Point of Use Transport System

Flash-Guard® systems are available in 5 sizes to accommodate the unwrapped method of sterilization using Pre-Vacuum or Gravity sterilizer.



Dimensions (Internal and External)

FG12

6.85" W
11.50" L
6.00" D

6.85" W
14.25" L
6.25" D

FG14

9.25" W
11.50" L
6.00" D

10.50" W
14.25" L
6.25" D

FG16

9.25" W
15.50" L
6.00" D

10.25" W
18.50" L
6.25" D

FG20

11.50" W
19.50" L
6.00" D

13.00" W
22.00" L
6.25" D

FG23

12.00" W
22.50" L
6.00" D

13.50" W
23.00" L
6.50" D



**MILLENNIUM
SURGICAL**

SURGICALINSTRUMENTS.COM

UNMATCHED SERVICE · UNLIMITED INSTRUMENTS

822 Montgomery Ave #205
Narberth, PA 19072
(800) 600-0428

Flash-Guard® Sterilizer to Point of Use Transport System

A full line of accessories is available including the Flash-Guard® monitor holder, cart, gloves, dome lids (for FG20 and 23 series only), baskets and inserts. For more information, contact a Product Consultant.

DOME LIDS



AVAILABLE ONLY ON
FG20 & FG23 SIZES

SECURED LIDS



TRANSPORT SECURELY

